
seca 717A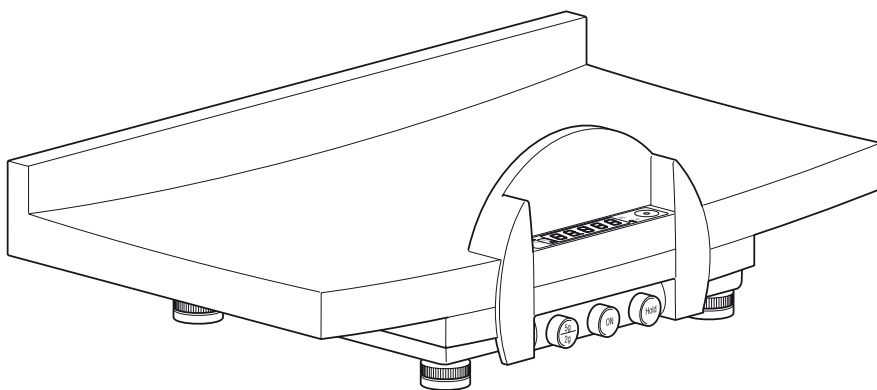

D	<i>Bedienungsanleitung und Garantieerklärung</i>	3
GB	<i>Instruction manual and guarantee</i>	13
F	<i>Mode d'emploi et garantie</i>	23
I	<i>Manuale di istruzioni e garanzia</i>	33
E	<i>Manual de instrucciones y garantía</i>	43
DK	<i>Betjeningsvejledning og garantibevis</i>	53
S	<i>Bruksanvisning och garanti</i>	63
N	<i>Bruksanvisning og garantierklæring</i>	73
FIN	<i>Käyttöohje ja takuu</i>	83
NL	<i>Bedieningshandleiding en garantieverklaring</i>	93
P	<i>Instruções de utilização e declaração de garantia</i>	103
GR	<i>Οδηγίες χειρισμού και εγγύηση</i>	113

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Full certification

seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products comply with European directives, standards and national laws. With seca, you are buying the future.

M

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol comply with the following directives and standards:

1. Directive 90/384/EEC governing non-automatic weighing instruments
2. Directive 93/42/EEC governing medical devices
3. DIN EN 45501 governing metrological aspects of non-automatic scales

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the development, production, sales and service of medical scales and measuring systems and ergometry.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

1. Congratulations!

In the **seca 717A** electronic baby scale you have acquired a highly-accurate yet simultaneously sturdy piece of equipment. seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

The **seca 717A** electronic baby scale is designed for use in a doctor's surgery or hospital. It serves both to determine

weight at birth and to check weight gained during growth.

Its maximum capacity is 15 kg, the display has a resolution of 5 g.

A special feature of this scale is that you can switch between two weighing ranges. Resolution in the lower weighing range is 2 g.

In addition, the scale has a HOLD function to store the weight determined and a TARE function so that additional weights such as cloths or pads are ignored for weighing.

The scale has Class 3 CE approval.


2. Safety

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the safety instructions in the instruction manual.
- Use only the mains unit supplied.
- Do not drop the scale or subject it to violent shocks.
- Have repairs carried out exclusively by authorised persons.
- Have scale serviced and re-calibrated on a regular basis (see chapter entitled "Servicing/recalibration" on Page 19).

3. Removing transport securing device

- Turn over the scale so that the base-plate faces upwards.
- Unscrew both screws out of the base of the scale and take off the transport securing device.
- Turn the scale back up the right way.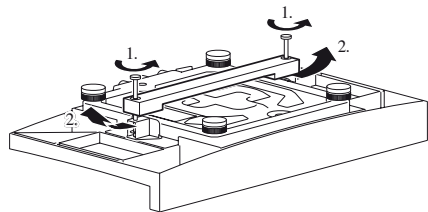

Important!


Keep the transport securing device in a safe place, as the scale has to be secured again to be transported any distance (see section entitled "Transport" on Page 18).

4. Power supply

The scale is powered by a mains unit. The connection socket for the mains unit is in the base of the scale.


Please use only the mains unit supplied or mains units from the seca range of accessories. Before plugging the unit into the socket, check whether the mains voltage marked on the mains unit matches local mains voltage.

- Turn over the scale so that the base-plate faces upwards.
- Connect the latching plug of the mains unit to the connection socket in the base of the scale.
- Then turn the scale the right way up.
- Connect the mains unit to a mains socket.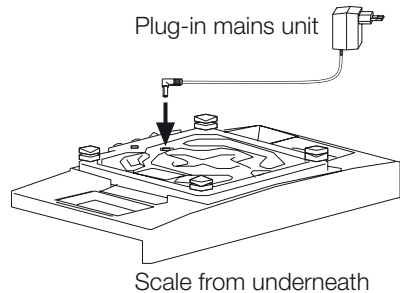

5. Set up and level scale

- Place scale on a firm surface.
- Undo the locknuts over the four screw feet.
- Level the scale by turning the screw feet. The bubble in the spirit level must be in the exact centre of the circle.
- Then tighten up the locknuts again
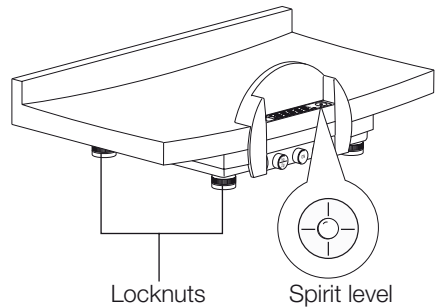


Important!

Every time it is moved, the base of the scale must be checked to see if it is level and corrected if necessary.

6. Operation

Controls and displays

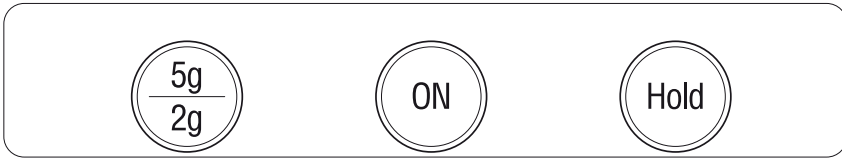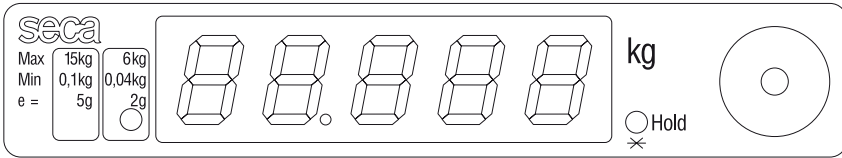
How to weigh correctly

- Press the green **ON** key.

SECA, 18:88:8 and **0.000** appear consecutively in the display. The scale is then automatically set to zero and ready for use.

- Place the baby on the scale.
- Read the weighing result off the digital display.

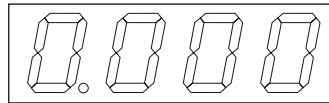The display has a resolution of 5 g. In the weighing range up to 6 kg, you can switch to a resolution of 2 g.

Switching weighing ranges

When the scale is switched on, it automatically uses the weighing range up to 15 kg. If the baby weighs less than 6 kg, you can switch weighing ranges and have a resolution of 2 g.

- Press the **5g/2g** key to change weighing range.

When the weighing range up to 6 kg with a resolution of 2 g is switched on, the **2g** display lights up.


Automatic switch-off/energy-saving mode

If no load is placed on the scale for 10 minutes, it switches off automatically.

Exceeding the weighing range

- If a load of over 15 kg is placed on the scale, or in weighing range 2, a load of over 6 kg, **StOP** appears in the display.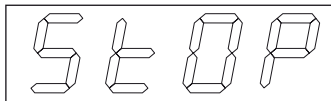

Taring weight display (TARE)

The tare function is useful if an additional weight (such as cloths or a pad on the weighing surface) are to be ignored in the weighing result. Proceed as follows:

Important!


The maximum initial load is up to 3 kg, or up to 1.2 kg in the weighing range to 6 kg. If a measuring rod is attached, ensure that maximum initial load is reduced by 400 g.

- Place the additional weight on the scale.
- Press the green **ON** key.

SEcR, **18:88:8** and **0000** appear consecutively in the display. The scale is then automatically set to zero and ready for use.

You can now perform as many weighing operations as you like with this additional weight.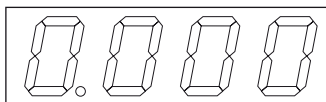


The Tare function can be reactivated at any time with a changed load or without any additional load at all.

- Place the new additional weight on the scale and press the **ON** key.

If **----** appears in the display, the additional weight has probably been removed. In this case, press the **ON** key.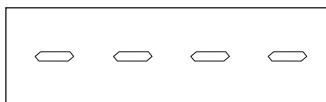


Storing the weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to put the baby down safely again before noting the weight.

- Press the **HOLD** key whilst the load is on the scale. The HOLD indicator flashes.

The value is stored in the display until the scale switches off automatically after 10 minutes.

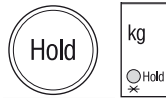- The HOLD function can be switched off by pressing the **HOLD** key again.

0.00 appears in the display with no load on the scale or the current weight. The "HOLD" display disappears.

Movements are damped by a special damping system. However, if the baby should move about on the scale too much, the HOLD function is not activated and the value displayed flashes. This prevents inaccurate measurements.

7. Cleaning


Clean the tray and the housing as required with a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

8. Transport

Before transporting the baby scale any distance, you should attach the transport securing device. This protects the highly-accurate electronics.

Proceed as follows:

- Turn the scale over so that the base-plate is on top.
- Attach the transport securing device and push the spacers between the frame and the tray support.
- Tighten up the screws on the base of the scale.

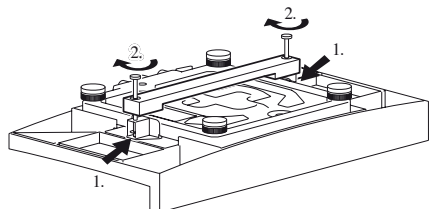Warning! Only tighten up the screws by hand, do not use a tool!

- Then turn the scale back up the right way.

9. What do I do if...

... no weight is displayed with a load on the scale?

- Is the scale switched on?
- Check the mains unit connection.

... 0.00 does not appear before weighing?

- Press the Start key again – the scale may be loaded with a maximum of only 3 kg/1.2 kg.

... - - - - appears before weighing instead of 0.00?

- Press the Start key again – the scale may be loaded with a maximum of only 3 kg/1.2 kg.

... the weight display is flashing?

- The HOLD function has not been activated because the baby is moving around too much on the scale. Press the **HOLD** key again.

... one segment is illuminated permanently or not at all?

- That point has a fault. Inform your service agent.

... 5tOP appears in the display?

- Maximum capacity has been exceeded - remove load from scale.
- Switch to the weighing range up to 15 kg if necessary.

... EEEEE appears in the display?

- Has the scale been started with the transport securing device in place? Remove the transport securing device (see section entitled "Removing transport securing device" on Page 14).
- A load far exceeding the permissible weighing range has been placed on the scale (see also "Exceeding the weighing range" on Page 17).
- A fault has occurred in the electronics. Inform your service agent.

10. Servicing/recalibration

Have the scale recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0109 (Hessen Weights & Measures Office). Recalibration is essential if one or more security marks are damaged.

We recommend having your scale serviced by your local service agent before having it recalibrated. seca Customer Service will be pleased to give you further assistance.

11. Technical data

Dimensions

depth: 320 mm
width: 551 mm
height: 163 mm
approx. 6.5 kg
+10 °C to +40 °C

Weight

Temperature range

Approval as per

Directive

90/384/EEC

Class III medical approval

Maximum capacity

Weighing range 1 15 kg

Weighing range 2 6 kg

Minimum capacity

Weighing range 1 0.1 kg

Weighing range 2 0.04 kg

Graduations

up to 6 kg: 2 g

up to 15 kg: 5 g

Accuracy at first calibration

Weighing range 1

0 up to 2.5 kg: ± 2.5 g

2.5 kg up to 10 kg: ± 5 g

10 kg up to 15 kg: ± 7.5 g

Weighing range 2

0 up to 1 kg: ± 1 g

1 kg up to 4 kg: ± 2 g

4 kg up to 6 kg: ± 3 g

EN 60 601-1:

insulated equipment, protection class II, electrical medical equipment, Type B

Medical product as per Directive

93/42/EEC

Class I

12. Spare parts and accessories

Spare parts

Plug-in mains unit

230 V~/50 Hz/12 V=/130 mA

seca no.: 68-32-10-252

Switchmode power adapter

100-240V~/ 50-60Hz / 12V= / 0.5A

seca no. 68-32-10-265

Accessories

seca 231 measuring rod for baby scales

Measuring range: 350 – 800 mm
14 – 32"

Graduations: 1 mm
1/8"

Weight: 2 kg

Item no. 231 1717 009

seca 429 head and foot support for baby scales

Item no. 429 0000 009

13. Disposal

Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national regulations which apply in your case.

For further information, contact our service department at:

service@seca.com

14. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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