Air OX • TecnoHeart • Monitor MD • Vital Test • CMS

Primo Soccorso
First Aid

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The First Aid project

The company Tecno-Gaz S.p.A., always committed to manufacturing first aid equipment, has worked out the FIRST AID PROJECT. This is a complete program to help you organize your business so as to make it efficient, conforming and ready to tackle every emergency condition.

Outpatient facilities must and are obliged to organize emergency plans with instruments and devices that are adequate to ensure appropriate first aid. The basic professional equipment should include the following:

- A SEMIAUTOMATIC DEFIBRILLATOR DAE
- A MULTIPARAMETER MONITOR or PULSE OXIMETER
- AN ELECTRONIC SPRINKLER
- RESUSCITATION SET
- DRUGS

Tecno-Gaz S.p.A. can supply the equipment above as well as support services and personnel training and information.

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Cardiac Arrest means an immediate interruption of the heart pumping functions, which leads to the patient's death if it is not treated in a suitable and timely manner.

There are many factors that can cause a cardiac arrest, but in most cases it is due to ischemic heart disease.

The patient is subject to a cardiac and respiratory arrest in the following cases:

- 1• When s/he is unconscious;
- 2• When s/he cannot breathe spontaneously (or gasps);
- 3• When the pulse of the carotid artery cannot be detected.

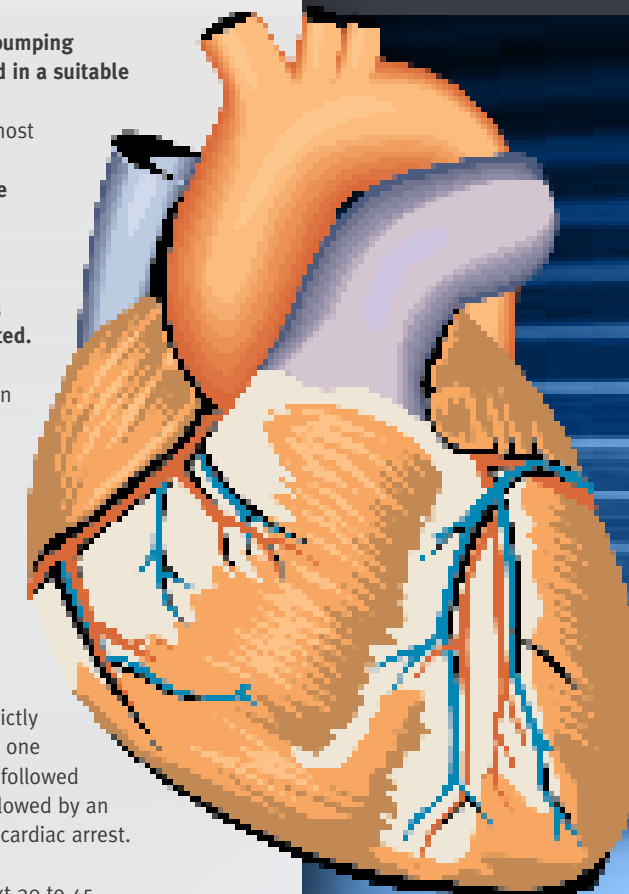
Items 2 and 3 must not be present simultaneously: only one can start the **BLS (Basic Life Support) sequence**.

Although the term "cardiac arrest" is often used as a synonym for sudden death, it is better to use it when a sudden loss of the heart functionality is experienced. Cardio-pulmonary resuscitation (CPR) manoeuvres may help the cardiac arrest to recede and give life back to the patient, if they are performed in a timely and correct manner.

Usually the word "cardiac arrest" is used, although it would be more correct to talk about **Cardiac and Respiratory Arrest (CRA)** as the respiratory and cardio-circulatory functions are strictly interconnected; when one of the two functions stops, the other one too is rapidly interrupted. An initial circulatory arrest is usually followed by a respiratory arrest and vice versa, a respiratory arrest is followed by an invariably rapid deterioration of the heart activity bringing to a cardiac arrest.

The sudden interruption of the circulation is followed in the next 30 to 45 seconds by respiratory gasping, apnea and start of pupil dilation. In the subsequent 45 seconds the pupils dilate completely.

When the primary event is a respiratory arrest, blood becomes progressively poorer in oxygen and in a few minutes the patient experiences asystole or ventricular fibrillation, which brings to an interruption of the heart pumping activity. This is why resuscitation measures for each of these two emergency situations must include both ventilation and circulation.

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Automatic and semiautomatic defibrillators

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- TecnoHeart S (semiautomatic)
- TecnoHeart A (automatic)

The semiautomatic TecnoHeart S system:

The AED defibrillators currently on the market are mainly semiautomatic. They are therefore controlled by vocal commands and the shocks are delivered by means of a manual command given by the operator. A simple guided operation.

The automatic TecnoHeart A system:

The Tecnoheart AED automatic defibrillators are the maximum expression of versatility. These models manage all of the operations autonomously, informing the care giver of the activities being performed by the machine. The shocks are also delivered independently, subsequent to a verbal warning, without operator intervention. This avoids delays and hesitation, and all activities are carried out without human involvement.

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Guided vocal instructions

Following the text messages and the vocal instructions, practically anyone can use this life saving device. The instrument automatically gauges when a potentially life-saving shock is required and protects the user from accidental discharges. Using a simple approach, the Tecnoheart device guarantees a response that is timely and without hesitation. Thanks to its compact dimensions, the care giver can use this machine rapidly and immediately in the event of an unexpected cardiac arrest.

Wide variety of accessories

AED devices are being increasingly installed in a wide variety of places and the Tecnoheart models provide an excellent example, thanks to the assistance they provide care givers, enabling them to respond rapidly to an emergency. Tecnoheart devices are battery-operated, with long-life, high performance lithium batteries that do not require maintenance

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“Emergency in outpatient facilities is definitely a very important and delicate aspect. The doctor must minimize the risk by identifying the patient's history in a complete and meticulous manner and associating it to a preventive diagnostic assessment. Besides this, every facility should have suitable equipment such as a defibrillator, a resuscitation set and other devices that can guarantee safe and professional interventions. Guaranteeing safety is a deontological and moral must for every professional.”

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Tecnoheart reliability

- Daily self-testing of all critical components, to ensure the equipment's constant efficiency
- Daily self-testing of batteries, circuits and software to ensure that efficacy is maintained at all times
- Weekly self-testing for the purpose of checking partial and full energy
- Electrode presence and functionality test. This is an extremely important check that only exists on a few, highly advanced AED defibrillators
- Battery capacity level – the charge level of the batteries is always visible, thereby permitting controlled management of the same
- Visible and acoustic status indicator

Tecnoheart ease of use

- Push-buttons for operation: only 1 button to be activated in response to the vocal command (semiautomatic version), no buttons (automatic version)
- Pre-connected electrodes: always ready for use
- Non-polarized electrodes
- Text display: a display duplicates the vocal commands and alarms through icons and written instructions
- Vocal prompts for the positioning of the electrodes and advanced CPR: to assist you in all kinds of activities and manoeuvres

Accessori | Accessories

Consolle per supporto a parete
Holder for wall-mounting
Tecnoheart holder with rear accessory pocket and expiry date inspection window

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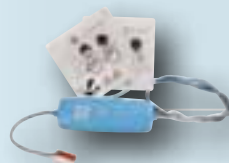
Tecnoheart technology

- Biphasic technology, more advanced system for delivering and supplying the shocks
- First shock efficacy: 100% certified
- Mean shock per patient: 2.3
- Time required for efficacious defibrillation: 55 secs

TecnoHeart Analysis algorithm

- Programmable VF/VT ratio: 120-240 bpm
- Asystole threshold: 0.08 Mv
- Detection of noise during analysis: yes
- Control of shocks delivered: yes
- Monitoring during CPR: yes
- Synchronized shock: yes
- Detection of Pacemaker pulse: yes

TecnoHeart Other features

- Internal memory for ECG data: 60 minutes
- Compatibility with manual defibrillators: yes
- Multi-event functionality: yes
- Pediatric electrodes: yes
- Product guarantee: 5 years
- Battery guarantee: 4 years

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Training version

A training version is available on request

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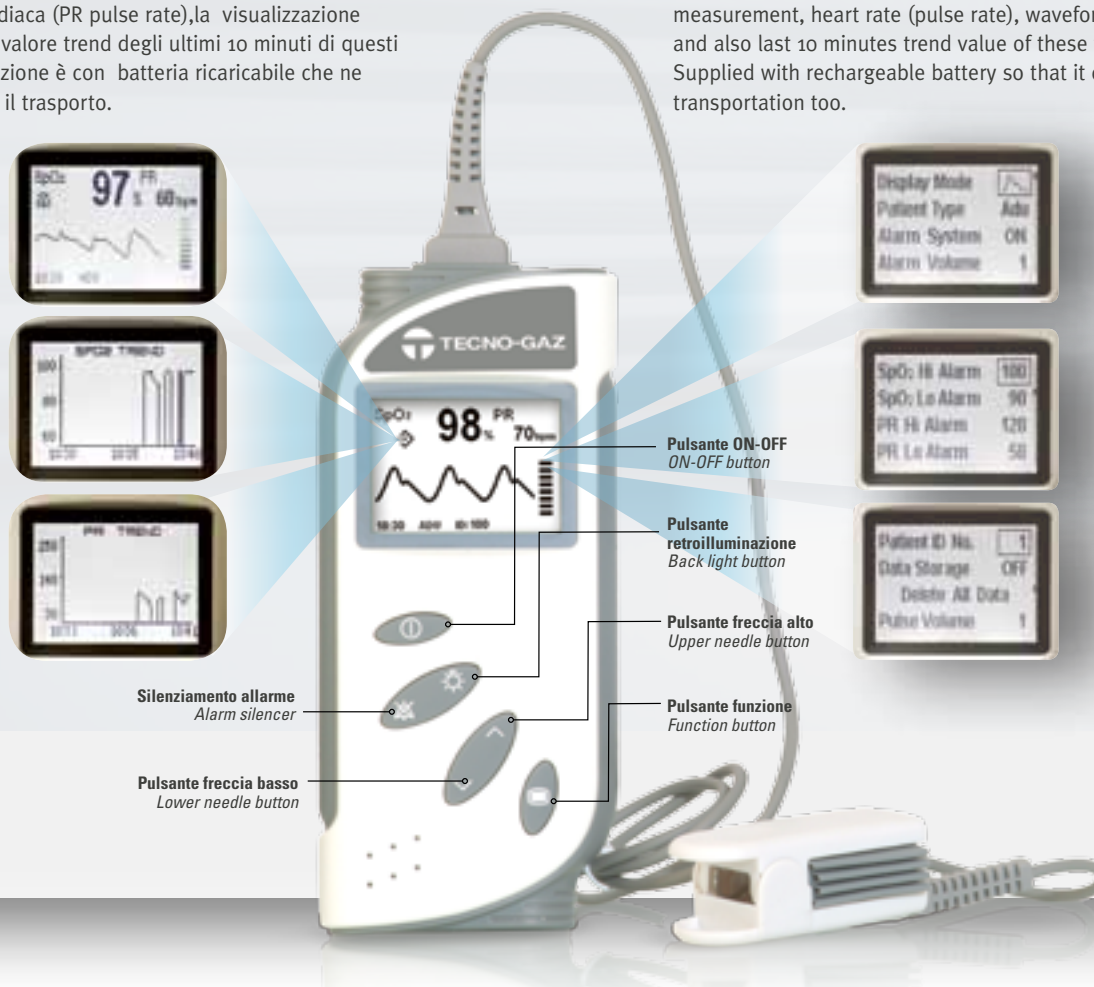
Pulse Oximetre Vital Test

VITAL TEST is Tecno-Gaz new and last generation pulse oxymeter which is distinguishable for its solidity, precision and advanced features. It's small, easy transportable, equipped with shoulder bag and easy to be used both by surgeons and assistants. The item is simple and professional, it can be used in intensive care units, surgeries, dentist's surgeries, emergency rooms, since it guarantees immediate, accurate and non-invasive monitoring. **VITAL TEST** has pleasant and ergonomic shape, it is easy to grip and to be transported, equipped with wide display and with simple and inductive controls. It allows SpO₂ measurement, heart rate (pulse rate), waveform visualization and also last 10 minutes trend value of these two parameters. Supplied with rechargeable battery so that it can be used during transportation too.

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Display

LCD wide display allows easy reading even at long distance, and it is also equipped with a control useful to activate or to remove retro-lighting in order to make it readable with all light conditions. Despite visualizing basic parameters it also visualizes battery level and alarms. Settings are the following:

- wave visualization or value decimal reading
- adults and children reading selection
- on/off sound and adjustable volume
- spO₂ min/max alarms
- pr min/max alarms
- patient's identification
- data reading
- volume pulsation

Optional

VITAL TEST can be supplied with proper support **code 11255**, planned to be fixed to dentist's unit, wall or trolley in order to allow a continuous vision even when hands are busy. An excellent solution both for dentists and outpatient surgeons. Interface program between computer and **VITAL TEST** can be supplied on demand in order to record and print all collected parameters. **VITAL TEST** is compatible with **Nellcor** and **BCI D89** sensors.

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Arrhythmia Review

Display of waveforms in real time and of vital signs with priority formulation, during and after the presentation of arrhythmic events.

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Tecno-Gaz has therefore made an electronic multiparameter monitor (MD) complete with accessories. The monitor has a double function: diagnosis and monitoring.

MD is a monitor that allows measurement of various vital parameters, which can also be checked singly.

Characteristics

- Measurement in adult, paediatric and neonatal patients.
- Management of the checks and controls of the monitor through front button panel and control knob.
- 10.4" TFT colour display for trace display.
- Analysis of arrhythmias, multi-derivation of the ST segment, pacemaker detection.
- OxyCRG function for control of infant circulatory and respiratory functions.
- 40-second review of waveforms.
- Graphic and numeric trends with 72-hour memory on all the measured parameters.
- Fold down carry handle on top.
- Analogue output for external VGA display.
- Equipped with adjustable minimum and maximum alarms, with visual and audible warning, and apnoea alarm.
- Operation from mains or integrated rechargeable batteries.
- Output for connection to centralised system.

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(art. 3002-E 5 litri senza carrello)

Oxygen is essential for cell survival and must be administered in cases where its concentration decreases due to a respiratory failure. Oxygen is indispensable in every emergency condition. For this reason every facility must be endowed with suitable equipment for correct oxygen administration.

Any outpatient resuscitation set must have precise features:

- rechargeable bottle of at least 1000 litres of gas delivering 15 litres/ min. for at least 60 minutes;
- reduction unit with litre dosing dispenser;
- reusable and autoclavable accessories with CE mark.

The resuscitator, model AIR OX item 3003-E , is a trolley-mounted resuscitator complete with a rechargeable oxygen bottle containing 1000 litres of gas, including:

- an oxygen reduction unit with pressure gauge, a flow meter complete with a 0-15 litre rotameter and an autoclavable humidifier in makrolon;
- a resuscitator bag in nitrile rubber complete with a unidirectional valve having a peep valve and integrated valve r eservoir connection;
- a shaped oronasal mask;
- a mouth opener;
- a tongue puller
- a Guedel tongue depressor;
- an oxygen connection tube.

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**Supporto
portamonitor**
*Monitor-carrying
support*

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Any emergency requires paying maximum attention to both organizational and operating aspects. With this view rationalization of the equipment is an essential aspect.

Tecno-Gaz S.p.A. has created a "Mobile First Aid Centre (CMS)".

This solution enables users to have a structure that features:

- a resuscitation set
- a defibrillator
- a drug kit
- a multi-parameter monitor

During any emergency speed is a primary factor for all first aid activities. For this reason, availing oneself of a mobile centre that is ready for use is undoubtedly the ultimate operating solution.

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In addition to the supply of a wide and qualified range of **FIRST AID** products, Tecno-Gaz S.p.A. also offers care plans and services to assist its customers.

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In addition, you can also request a copy of **EMERGENZE MEDICHE IN STUDIO ODONTOIATRICO** (MEDICAL EMERGENCIES IN THE ODONTOIATRIC ROOM) published by SEE.

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We organize BLS training courses that are I.R.C. certified and include issuing of a final certificate that is valid on the entire national territory.

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All rights reserved. Variations can be done without notice. Tecno-Gaz S.p.A. is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.