# dermLite® FOTO

3Gen

ENGLISH

DANSK

DEUTSCH

ESPAÑOL

FRANÇAIS

GREEK

ITALIANO

NEDERLANDS

PORTUGUÊS

SUOMI

SVENSKA


# Dermlite<sup>®</sup> FOTO QuickStart Guide

## ① DermLite FOTO37 - The Lighthead.

The DermLite FOTO37 lighthead attaches to cameras via a standard 37mm\* filter thread. The faceplate spacer can be removed for cleaning.


CAUTION: DO NOT AUTOCLAVE ANY PART.  
CAUTION: DO NOT LOOK DIRECTLY INTO THE BRIGHT LED LIGHT.  
CAUTION: UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

\*Optional stepping rings allow for attachment to different filter thread dimensions. Please contact 3Gen for availability of your desired size.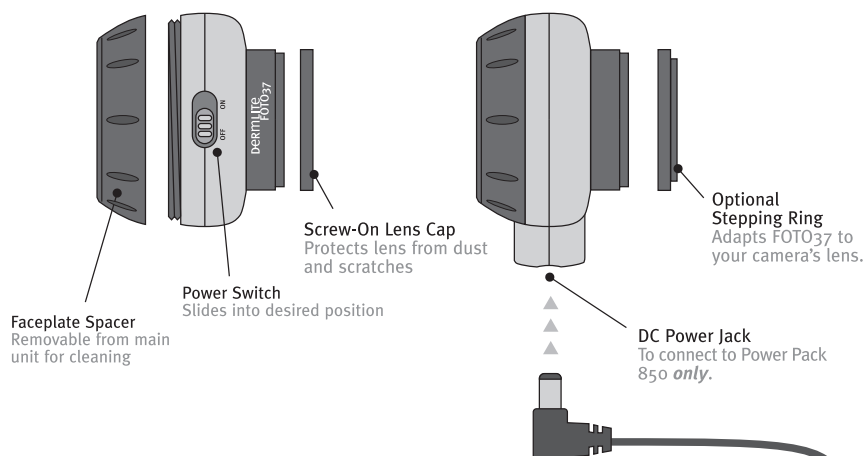


## ② Power Pack 850 - Compressed Energy.

This compact unit delivers about 2h of continuous energy to the FOTO37 lighthead via the 9" cable. To recharge, attach the Charger's cable to the Power Pack's DC power jack. The unit can be affixed to cameras horizontally or vertically by sliding it onto the camera's flash shoe adapter, or to the camera bottom by using 3Gen's Flash Foot Adapter.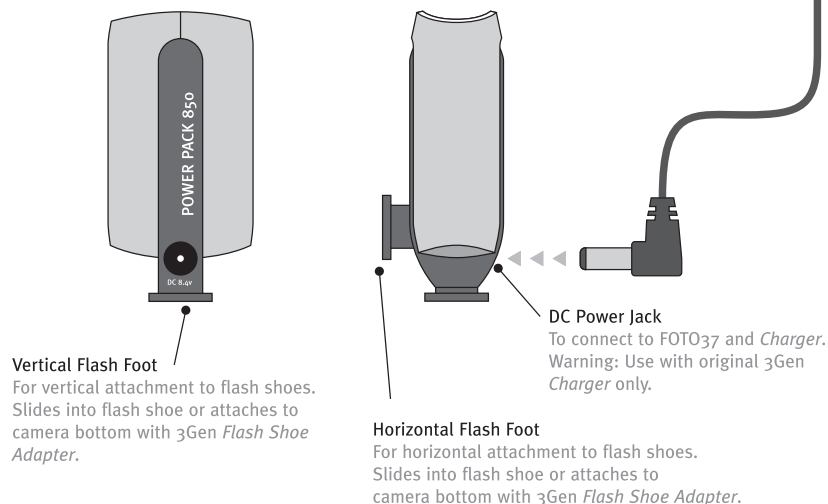


## ③ Flash Foot Adapter - A Practical Option.


The Flash Foot Adapter is designed to allow you to attach the Power Pack to cameras and camcorders without flash shoe compatibility. Simply slide the Power Pack into the adapter's flash shoe, and tighten the screw onto the camera bottom's tripod attachment.

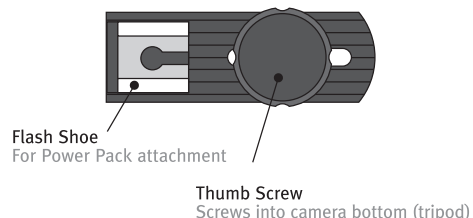
### 1 Adjust Thumb Screw For Power Pack attachment clearance
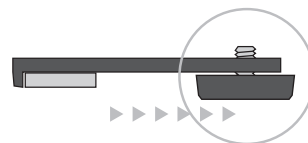
### 2 Slide Power Pack into Shoe For Power Pack attachment
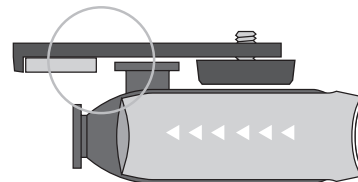
### 3 Attach to Camera Bottom

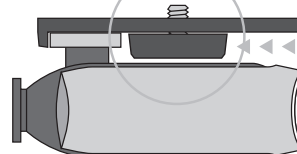3Gen<sup>®</sup>


## 4 Mounting Options

The DermLite FOTO system provides for a variety of Power Pack attachment configurations.

For mounting the unit onto a photographic camera with flash shoe, you may choose to mount the Power Pack as shown in option A.

Should you wish to use your product in combination with a camcorder or a camera without flash shoe compatibility, you may wish to mount the Power Pack with the help of the included *Flash Foot Adapter*, shown in option B.

NOTE: IT IS USER'S RESPONSIBILITY TO SELECT APPROPRIATE CAMERA/CAMCORDER AND CONNECT IT CORRECTLY FOR THEIR USE.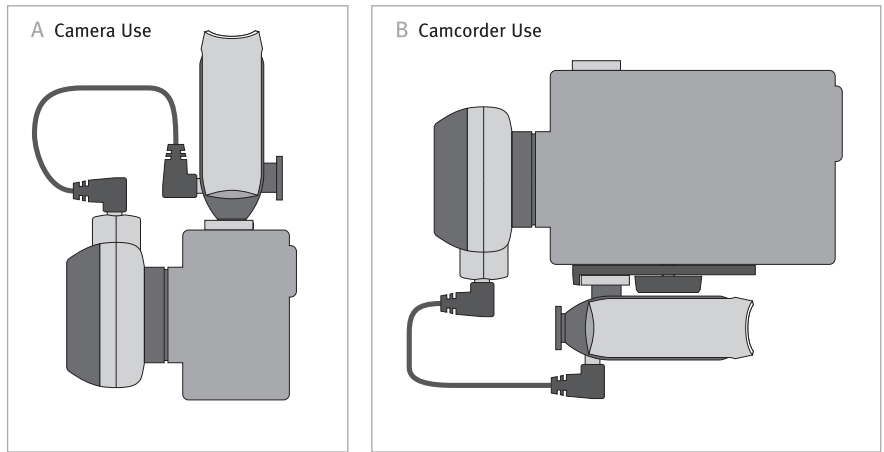

## 5 Charging the Power Pack

**CAUTION: FOR SAFETY REASONS, NEVER ATTEMPT TO CONNECT THE POWER SUPPLY DIRECTLY TO THE POWER PACK.**

**CAUTION: NEVER ATTEMPT TO CONNECT THE POWER SUPPLY DIRECTLY TO FOTO LIGHTHEAD.**

The Lithium-Ion Power Pack should be recharged when not in use and with the supplied 3Gen Charger only. Plug the charging cord into the Power Pack, and connect the universal power supply to the Charger.


The universal power supply can be fitted with an international 220V plug, using the tool provided.

Tip: You may want to keep the Charger on your desk, or within easy reach, and keep the power supply out of sight.

**CAUTION: DO NOT RECHARGE POWER PACK WHILE IN USE. ONLY RECHARGE WITH ORIGINAL 3GEN CHARGER.**

**CAUTION: WHENEVER WORKING WITH ELECTRICAL COMPONENTS, ALWAYS CHECK CABLES AND COMPONENTS FOR VISIBLE SIGNS OF WEAR. DO NOT HANDLE ELECTRICAL COMPONENTS THAT SHOW SIGNS OF WEAR.**

WARRANTY: 5 YEARS FOR PARTS AND LABOR

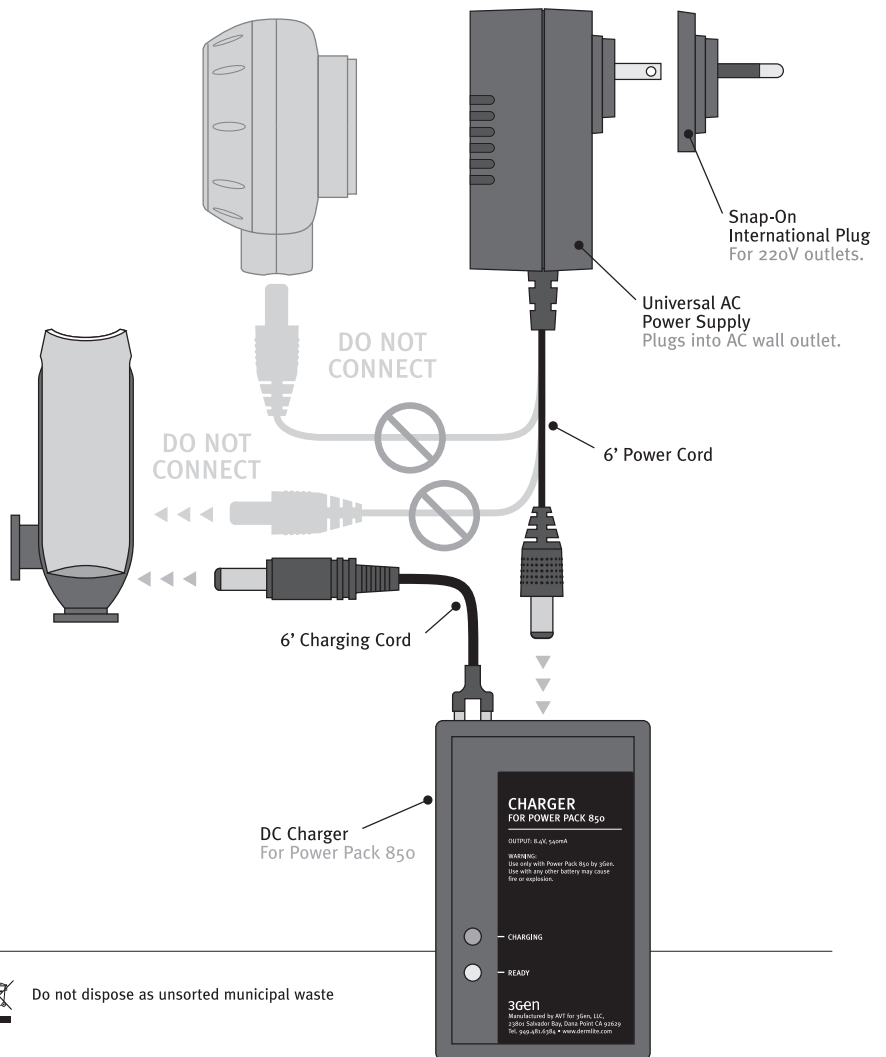Symbol	Explanation
REF	Model Number
SN	Serial Number
	Attention, consult accompanying document
	Manufacturer
	EC Representative (EU Only)

Do not dispose as unsorted municipal waste

M. Devices Group,  
Marlborough House, Riding Street,  
Southport, PR8 1EW, UK.  
Tel: +44 1704 544 944 Fax: +44 1704 544 050

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