Biopsy Container.

contains: 4% Formaldehyde


EN INSTRUCTIONS FOR USE
DA BRUGSANVISNING
DE GEBRAUCHSANLEITUNG
ES INSTRUCCIONES DE USO
IT ISTRUZIONI PER L'USO
FI KÄYTTÖOHJEET
FR MODE D'EMPLOI
NO BRUK
PT INSTRUÇÕES DE UTILIZAÇÃO
SE BRUKSANVISNING
GR ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
NL GEBRUIKSAANWIJZINGEN
RUS ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
TUR KULLANIM KILAVUZU
PL INSTRUKCJE UŻYTKOWANIA

2021.03.16

Manufactured in Denmark for
 BiopSafe ApS
 Carollinevej 2, 1TV, DK-2900 Hellerup
 Denmark www.biopsafe.com

© 2014 BiopSafe ApS. All Rights Reserved.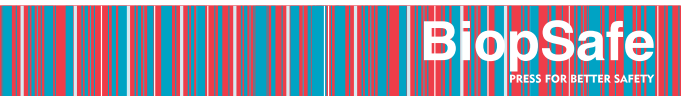

EN

Intended Use
 Preservation of biopsies until medical analysis.

Contents
 4% formaldehyde in aqueous solution (10% formalin solution) and <2,5% methanol.

DANGER
 Contents: Formaldehyde, methanol.
 May cause an allergic skin reaction.
 Suspected of causing genetic defects.
 May cause cancer.
 Obtain special instructions before use.
 Use personal protective equipment as required.

Storage temperature
 18-30°C incl. max. 15 days at 2-18°C

Expiry date
 Refer to the BiopSafe® label

Procedure for sampling

1. Wear protective medical gloves (Butyl or Nitrile).
2. Unscrew the lid.
3. Take the biopsy and place it in the container.

4. Close BiopSafe® by screwing the lid onto the container. The lid must be screwed all the way on!
5. Release the formalin by pressing the top of the lid. Check that the biopsy is completely exposed to the formalin.
6. Put a label with patient data and other relevant information on the BiopSafe® according to local procedures.
7. Send the BiopSafe® for analysis according to local procedures for the shipment of diagnostic samples.

Procedure for analysis

1. Wear protective medical gloves (Butyl or Nitrile).
2. Open BiopSafe® in a laminar flow cabinet with exhaust.
3. Unscrew the lid.
4. Take out the biopsy for analysis.
5. The formalin is discarded according to local procedures.
6. The BiopSafe® is discarded according to local procedures.

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