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1 - Pour the necessary quantity of lukewarm water into a container (35°C) (figure 1) following the instructions in the table below (figure 2). You can use tap water.

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Figura 2

For cold sterilization 2% SOLUTION	
Litres of solution to be prepared	Quantity of STERIL-C to be added
0,5 LITRES	10 grammes
1 LITRE	20 grammes
2,5 LITRES	50 grammes
5 LITRES	100 grammes
For primary decontamination and disinfection 1% SOLUTION	
Litres of solution to be prepared	Quantity of STERIL-C to be added
1 LITRE	10 grammes
2 LITRES	20 grammes
5 LITRES	50 grammes
10 LITRES	100 grammes

Figure 2

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2 - Lift the tab (figure 3) and pull it around the entire circumference of the cover (figure 4) up to remove it.

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To open the canister push up the protruding part of the cover (figure 5).

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3 - Fill the measure (figure 6) which is inside the canister.

On the measure some horizontal lines are marked:

- pour the powder up to the line which marks 15 cc to get 10 grammes of powder;
- pour the powder up to the line which marks 30 cc to get 20 grammes of powder.

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4 - Pour the necessary quantity of **STERIL-C** powder into the container (figure 7) according to the quantity indicated in the table (figure 2) and mix (figure 8) carefully for 2-3 minutes getting to the bottom of the container.

The solution is ready to be used after 5 minutes.

For the treatment of fiber-optic devices it is preferable to use the solution 15 minutes after its preparation.

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